



1. Draw back the plunger to the 1cc mark (if using 0.5cc of Ethoss®TCP) or to the 2cc mark (if using 1cc of Ethoss®TCP). Do not remove the cover at this stage.




4. Fill with sterile saline (0.9% Sodium Chloride for injection) to the top of the syringe.



2. In order to loosen the powder and aid mixing, hold the syringe horizontally and tap the syringe until the Ethoss®TCP spreads out.



5. Tap the syringe for 10 seconds until the fluid has visibly soaked into the powder.

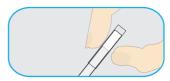

3. Return the syringe to the vertical position with the plunger at the bottom. Remove the end cover.



6. Place a sterile gauze over the end of the syringe and depress the plunger to expel any excess fluid. Your Ethoss®TCP is ready to use.